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Ziegelei 1

D-72336 Balingen

E-Mail: [info@kern-sohn.com](mailto:info@kern-sohn.com)

Tel: +49-[0]7433- 9933-0

Fax: +49-[0]7433-9933-149

Internet: [www.kern-sohn.com](http://www.kern-sohn.com)

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## Mechanical height measuring rod

### 1 Technical data

KERN (type)	TMBA-A01-A
Model	MBA-A01
Measuring range	30 – 80 cm
Readability	1 mm
Tolerance	5 mm
Dimensions (D x H x W) mm	850 x 240 x 60 mm
Net weight	approx. 800 g

### 2 Declaration of conformity

The current EC/EU Conformity declaration can be found online in:

[www.kern-sohn.com/ce](http://www.kern-sohn.com/ce)

---

### 3 Appliance overview

---

1. Measuring rod
2. Head stop
3. Caliper gauge/Heel stop

---

## 4 Basic Information (General)

---

⇒ Carefully read this operation manual before setup and commissioning, even if you are already familiar with KERN balances.

### **Safety Instructions**

- Observe operating instructions.
- Please use no other than the supplied fixtures for mounting.
- When moving the height measuring rod ensure that the person to be measured cannot be injured.
- The body height measuring emits only reliable values, when the back and the head are straightly aligned, nevertheless the body height of a person may change about some cm during a day.

### **Test device control**

- We recommend a metrological examination of the accuracy of the measuring rod, however, this is not mandatory as the determination of human body height involves rather large, intrinsic inaccuracies.

---

## 5 Unpacking/installing

---

### **Testing upon acceptance**

Please check packaging as well as the height meter on receipt for possibly visible external damages.


### **Packaging/ Return**

Keep all parts of the original packaging for a possibly required return. Only use original packaging for returning.


---

## 6 Body height measurement


---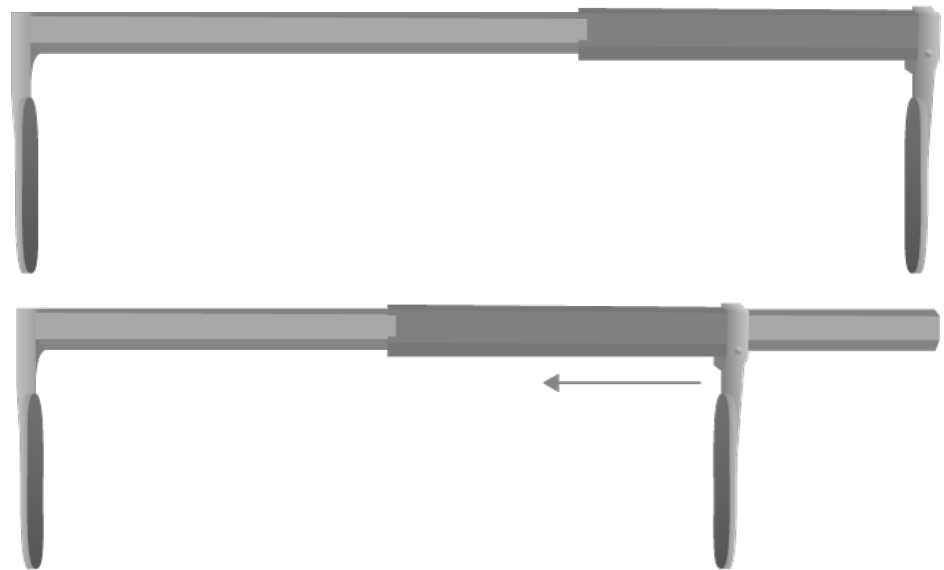

**Fixed stop**  
min. 30 cm

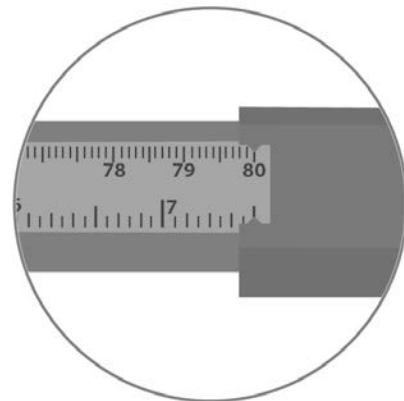**Movable stop**  
max. 80 cm

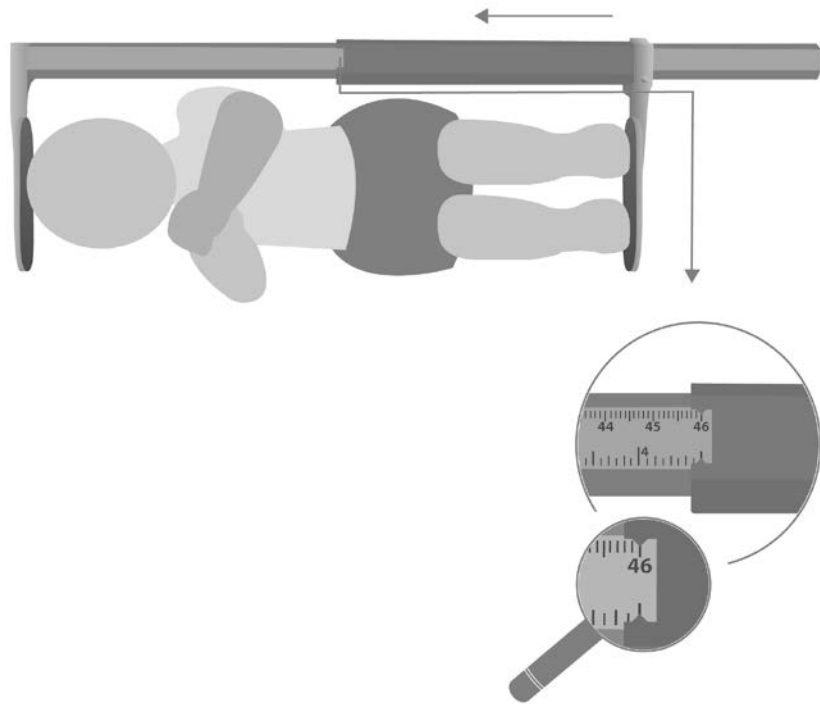
- ⇒ Position the baby in a way that the head touches gently the fixed stop.
- ⇒ Shift caliper gauge acc. to fig. till the heel stop.
- ⇒ Read the body height.

---

## 7 Cleaning, Maintenance, Disposal

---

<b>Cleaning</b>	Clean the height measuring rod with household detergent or commercially available disinfectant, e.g. 70% isopropanol. We recommend a disinfectant suitable for wiping disinfection. Please follow manufacturer's instructions.
<b>Disinfection</b>	Do not use abrasive or aggressive cleaners such as spirits or alcohol or similar as they might damage the high-quality surface.
<b>Sterilisation</b>	Sterilisation of the appliance not allowed.
<b>Disposal</b>	Disposal of packaging and height meter must be carried out by operator according to valid national or regional law of the location where the appliance is used.

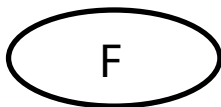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